


GINGI - BLOC ●●

Gingival Protective Barrier

Methacrylate
Filler and Dye
Photo Initiator

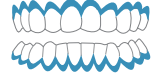

PRODUCT DESCRIPTION:

D-Tech Gingi - Bloc is a light curing resin based gingival barrier in syringe packing for isolating gum and neighbouring teeth. It's methacrylate based formulation is designed with adequate strength to maintain a barrier and yet it can be easily removed from gums, interdental spaces and undercuts.

FORMULATION:

Methacrylate
Filler and dye
Photo Initiator

FEATURES:

1. Light cured resin - based barrier.
2. Protects soft tissue / gums with impervious seal.
3. Easily removable after bleaching or other dental treatment.

INDICATIONS FOR USE:

It is indicated to protect the gingival tissue in cases where there is a risk of irritation by contact with products used in the clinical procedure, such as dental bleaching with Hydrogen peroxide at 35%.

ADVANTAGES:

1. The product is sufficiently viscous and thixotropic to allow easy application and control over the place to be protected without running.
2. Minimum and perfect adhesion on the gingiva so that it is easily detached and removable.
3. No gingival tissue irritation
6. Opaque blue colour , which allows the contrast to

DIRECTION OF USE:

Step 1- Apply Gingi-Bloc barrier resin on the maxillary arch along the gingival margin around each tooth to be whitened.

Step 2- Apply Gingi -Bloc barrier resin on the mandibular arch to ensure patient comfort.

Step 3 - Ensure the soft tissue of the gingival margin is completely covered. Light cure in a sweeping motion until gingival barrier is hardened.

Step 4- Use an appropriate dental tool to remove Gingi -Bloc barrier resin for example Dental Bleaching.
Rinse thoroughly.

CONTRA INDICATIONS:

No known contra indications exist for this product. No known systemic allergy is recorded. However , the practitioner is advised to observe the patient carefully for any signs of allergy.

PRECAUTIONS:

P280 Wear protective gloves/protective clothing/eye protection/ face protection.

PACKAGING:

- 1) Black (Light safe) Syringe of 1 gm Gingi - Bloc
- 2) 2 Applicator Tips

1 x 1 g syringe



1 x 2 Applicator Tips




STORAGE:

Store in cool dark place. Keep away from direct light and heat .

DISPOSAL:

Refer to community provisions relating to waste disposal. In their absence , refer to national or regional provisions relating to waste disposal.

For Professional use only in the practice of dentistry

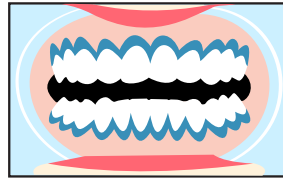

AVOID CONTACT WITH SKIN, EYES
STORE IN A COOL AND DRY PLACE



D Tech Dental Technologies,

62, Reality Warehousing Gat No 1337/1, Wagholi
Pune India- 412207 Customer Care- 9370145806
E Mail – info@dtechasia.com

CE 0123

EC REP OBELIS S.A
Bd General Wahis, 53 1030 Brussels
Belgium, E mail: mail@obelis.net

For use only in dentistry

d-tech

www.dtechasia.com