



### Check the fitting

Check the fitment of printed C&B on a corresponding model.

If it fits properly go to the next step



### Polishing & Finishing

Polish with a rotary tool/brush applying light to moderate pressure



### Glaze and Post Cure Glaze

Apply the D Tech Pro Glaze material evenly on the printed object using a micro brush. Cure 10-15 seconds in a curing chamber.

Dip the object in Glycerine (50%) + water (50%) mixture and cure it for another 30-40 seconds.



### Second Polish

Polish again after the Glaze is fully cured. Polish again with a rotary tool/ brush applying light to moderate pressure.