

# GP - Solv

## Root canal filling remover



www.dtechasia.com

CE 0123

### 1. Product Description & Main Ingredients

GP-Solv is a "Chloroform Free" solvent for effectively softening or dissolving root canal filling materials including gutta percha and Zinc Oxide Eugenol paste during root canal retreatment.

#### Main Ingredients

Natural Solvent

### 2. Indications

D-Tech GP-Solv softens filling materials for easy removal, improving access to the root canal.

### 3. Contraindications

3.1 No Known Contraindication exists for this product no known systemic allergy is recorded. However, the practitioner is advised with copious amounts of water and consult a physician immediately.

### 4. Precautions (Please read carefully before use)

- 4.1 Contact with mucosa can cause mild irritation. In case of contact wash with copius qty of water.
- 4.2 In case of contact with eyes, flush immediately with water and seek medical attention.
- 4.3 A tooth treated with GP-Solv must be carefully dried before a new restoration made of Eugenol resin or acrylic, is placed.
- 4.4 Flammable Product - Do not place in direct contact with any heat source or flame.

### 5. Application Method

5.1 Place a small amount of GP-Solv in the canal using a soaked cotton pellet or a syringe. Wait approximately for 15 minutes. You can feel the gutta percha significantly softer. You may also take a small quantity of GP-Solv in a dappen dish & dip the point of the endodontic instrument in GP-Solv. Proceed with cleaning of the canal. If any resistance is felt remove the instrument. Clean & repeat the process. If cleaning the canal presents a lot of difficulty the "two visit method" is recommended.

#### 5.2 Two Visit Method:

Place a cotton pellet soaked in GP solvent on the

fillings & cover with a temporary sealant. Recall the patient two days later. You will find the filling softened and very easy to remove.

### 6. Packaging & Variants

- 1 Bottle of 10ml

### 7. Storage & Handling:

Store at a temperature of 4-25°C (39.2-77.0°F)  
(Shelf life: 2 years from date of manufacture).

REFERENCE NO: IFU/GP/04/0625

<b>D Tech Dental Technologies</b> Plot No. 118/119, Sub Plot No. 32, Lane No. 7, Ramtekdi Industrial Estate, Hadapsar, Pune - 411028. INDIA. Customer Care : 9370145806 E mail : info@dtechasia.com Url : www.dtechasia.com	 Storage	 Keep Dry	 Keep away from Sunlight	 Authorised Representative	 Manufacturer
 Non Sterile	 Medical Device	 Unique Device Identifier	 Instruction for Use	 Batch No	 Expiry
<b>Limitation of Liability</b> Except where prohibited by law, D Tech will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability					
<b>For use only by a dental professional</b>					

**European Representative OBELIS**  
Bd General Wahis, 53, 1030 Brussels, Belgium  
E mail : mail@obelis.net

**Last Revised: 23 June 2025 Rev 04**