

GINGI - BLOC

Gingival Protective Barrier



www.dtechasia.com



1. Product Description & Main Ingredients

D-Tech Gingi - Bloc is a light curing resin based gingival barrier in syringe packing for isolating gum and neighboring teeth. It's methacrylate-based formulation is designed with adequate strength to maintain a barrier and yet it can be easily removed from gums interdental spaces and undercuts.

Main Ingredients:

Methacrylate Monomer & Oligomer, Filler & Dye, Photo Initiator.

2. Indications

It is indicated to protect the gingival tissue in cases where there is a risk of irritation by contact with products used in the clinical procedure, such as dental bleaching with Hydrogen peroxide at 35%.

3. Contraindications

- 3.1 No known contra indications exist for this product. No known systemic allergy is recorded.

However, the practitioner is advised to observe the patient carefully for any signs of allergy.

4. Precautions (Please read carefully before use)

- 4.1 Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

- 4.2 In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

4.3 Important Notes -

- 4.3.1 Light devices should emit wavelength of about 450 nm and intensity should be regularly Inspected. The light should be placed as close as possible to the filling material.

5. Application Method

- 5.1.1 Apply Gingi-Bloc barrier resin on the maxillary arch along the gingival margin around each tooth to be whitened.

- 5.1.2 Apply Gingi-Bloc barrier resin on the mandibular arch to ensure patient comfort.

- 5.1.3 Ensure the soft tissue of the gingival margin is completely covered. Light cure in a sweeping motion until gingival barrier is hardened.

- 5.1.4 Use an appropriate dental tool to remove Gingi - Bloc by lifting it from one end. Rinse thoroughly.

6. Packaging & Variants

- 1 Syringe of 1g
- 2 Applicator tips

7. Storage & Handling:

Store in cool dark place. Keep away from direct light and heat.

(Shelf life: 2 years from date of manufacture).

REFERENCE NO: IFU/GB/04/1125



D Tech Dental Technologies

Plot No. 118/119, Sub Plot No. 32, Lane No. 7,
Ramtekdi Industrial Estate, Hadapsar, Pune -
411028. INDIA.
Customer Care : 9370145806
E mail : info@dtechasia.com
Url : www.dtechasia.com

European Representative OBELIS

Bd General Wahis, 53, 1030 Brussels, Belgium
E mail : mail@obelis.net

Last Revised: 11 Nov. 2025 Rev 04



Storage



Keep Dry



Keep away
from Sunlight



Authorised
Representative



Manufacturer



Batch No



Expiry



Non Sterile



Medical Device



Unique Device Identifier



Instruction for Use

Limitation of Liability

Except where prohibited by law, D Tech will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability

For use only by a dental professional